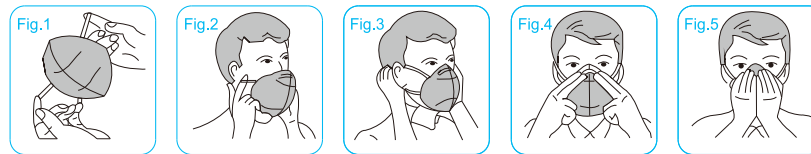
**Filtering Half Mask
FFP2 NR
SYMS-1**

Instruction for Use

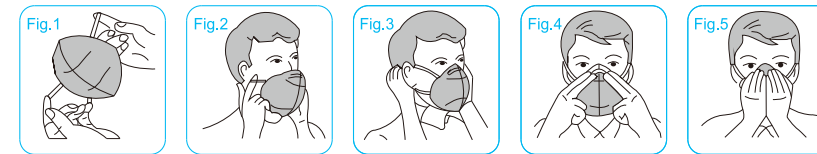
- Prior to wearing the face mask, inspect the face mask for damage and contamination. Verify all components of the mask such as ear loops, nose clip, etc. are in good condition.
- Stretch and unfold the mask, wear as shown.
- Completely cover the mouth, nose and lower jaw, press the noseclip tightly.
- Stretch the ear loops appropriately to adjust the tightness, making the mask and the face fit properly.
- Exhale sharply after completely cover the face, if air blows on your face or eyes, readjust the mask to ensure the seal.

Warning

- Use the mask in well-ventilated areas, where there is no lack of oxygen (> 19.5%) and where there is no explosive atmosphere.
- The mask must be positioned correctly in order to provide the expected level of protection. Do not use if facial hair (beard, sideburns) prevents the mask from being fully sealed, and leakage is detected.
- This mask DOES NOT PROTECT the user against gases or vapours and solvents from spray painting.
- Before use, please check whether the package is in good condition, the appearance of the mask should be free from damage, if damaged, do not use.
- This mask marked "NR", shall not be used for more than one shift.
- This mask is not suitable for users who have an ear that has been damaged.
- The user must have read the instructions before using it. Failure to observe these instructions for use may have adverse effects on the health of the user and expose him to serious or even life-threatening risks.
- Leave the area immediately if you experience breathing problems and/or dizziness or faintness.

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